

Mouthpiece

Mouthpiece is an accessory to the Breathbag for sampling of breath test intended for analysis with Kibion® Dynamic Base and Kibion Dynamic Pro devices. Mouthpiece consists of two conical tubes and a membrane preventing air leakage. *NOTE: do not disassemble.*

Mouthpiece is for single use only but can be reused by the same patient for additional sampling if required e.g. 0-sampling and test-sampling of Urea breath test.



Specifications

Product	Mouthpiece
Article No.	8007
Package size	25 pcs per package Every Mouthpiece is single wrapped
Storage and transport	Room temp./Ambient temp Protected from UV-light
Shelf life	3 years from manufacturing Expiry date is printed on the package.
CE-marking	Medical device class I CE-marked acc. to 93/42/EEC.

MANUFACTURER

Kibion GmbH
Haferwende 31
28357 Bremen
Germany
Contact
+49 (0) 421 27 86 5-0
Support.kibion@mayoly.com
www.kibion.com

Related products

Product	Article No.
Single Breathbag	8004
Double Breathbag	8005
Kibion Dynamic Base	8031
Kibion Dynamic Pro	8032

Mouthpiece

Intended use

Mouthpiece is intended to be used with Breathbag for sampling of breath test samples for analysis by Kibion Dynamic and IRIS analyzers.

The patient exhales through the Mouthpiece into a Breathbag under supervision of trained professional medical staff in a hospital or medical clinical.

Principle of operation

- Connect a Mouthpiece to the Breathbag tube prior sampling
- Exhale into the Breathbag through the Mouthpiece.
- Remove the Mouthpiece and close the tube with the blue stopper.
- Remove the blue stopper and connect the Breathbag to Kibion® Dynamic Base or Kibion Dynamic Pro unit for analysis.

Diabact® Urea Breath Test sampling



Exhale into the 0-sample Breathbag



Take a Diabact tablet with a glass of water.
Wait for 10 minutes



Exhale into the test-sample Breathbag

Kibion Dynamic Base and Pro

Breathbag connection

